


CRITICAL BATTERY RUN-IN AND CARE

December, 2016

SEE YOUR OWNER'S MANUAL FOR FULL INSTRUCTIONS

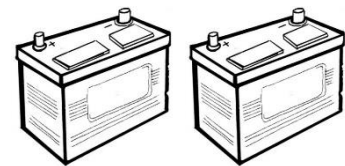

Initial Care

- Ensure the batteries are fully charged before the first use
- During the first 5 cycles, avoid excessive loads (e.g. steep slopes and beaches) and deep discharges
- Fully charge the chair (the charger light will turn green)

New batteries take longer to achieve full charge and the capacity is reduced; this will improve.

Recommendations

- Avoid allowing the batteries to get very flat
- Charge as soon as possible after flattening
- Always fully recharge the batteries (this may take more than 8 hours)

To get the maximum range from your batteries

- Ensure that the tyre pressures are correct for the weight and the terrain
- Avoid stop-go driving; try to maintain a constant speed
- Try to avoid inclines
- Limit baggage weight carried

PLEASE NOTE: The charger may not operate if the batteries have been allowed to go completely flat. If this happens, call your Magic Mobility Dealer for assistance.

Contact Magic Mobility if you require further information

3 International Court, Scoresby Vic 3179
Tel +61 3 8791 5600 Email sales@magicmobility.com.au
<http://www.magicmobility.com.au>