

# CONTROLLER LOCK/UNLOCK PROCEDURE

## Locking the control system


The Control System can be locked in one of two ways. Either using a button sequence on the keypad or with a physical Key. How the Control System is locked depends on how the wheelchair manufacturer has programmed the system.

### Keypad locking

TO LOCK THE WHEELCHAIR USING THE KEYPAD:

- While the control system is switched on, depress and hold the On/Off button
- After 1 second the control system will beep. Now release the On/Off button
- Deflect the joystick forwards until the control system beeps
- Deflect the joystick in reverse until the control system beeps
- Release the joystick, there will be a long beep
- The wheelchair is now locked.

The following screen will be displayed, the next time the Control System is switched on.

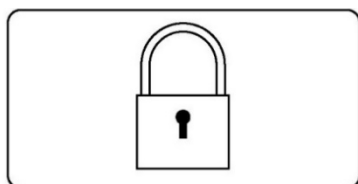

If an LED Joystick Module is fitted the Speed Indicator LEDs will ripple from left to right.

TO UNLOCK THE WHEELCHAIR:

- If the control system has switched off, press the On/Off button
- Deflect the joystick forwards until the control system beeps
- Deflect the joystick in reverse until the control system beeps
- Release the joystick, there will be a long beep
- The wheelchair is now unlocked.

### Key locking

TO LOCK THE WHEELCHAIR WITH A KEY:

- With the Control System switched on, insert and remove a R-Net supplied key into the Charger Socket on the Joystick Module. A short beep will be heard
- The wheelchair is now locked.

The following screen will be displayed, the next time the Control System is switched on.



TO UNLOCK THE WHEELCHAIR:

- If the control system has switched off, press the On/Off button
- Insert and remove a R-Net supplied key into the Charger Socket on the Joystick Module. A short beep will be heard
- The wheelchair is now unlocked.

### Video demonstration

Please refer to our video on YouTube for a visual demonstration:

[How to lock your JSM controller](#)

---

### Contact Magic Mobility if you require further information

3 International Court, Scoresby Vic 3179

Tel +61 3 8791 5600

Email [enquiries@magicmobility.com.au](mailto:enquiries@magicmobility.com.au)

<http://www.magicmobility.com.au>